

# eMedCheck - Anthrax

Blackberry App Walk Through

# Opening the App



After downloading and installing the application, you can find it in your device's download folder.

# Beginning a Screening



Once the device is opened, and the user continues past the splash screen, the household creation screen appears. Add household members to this screening with the “New Person” button. Then hit “Next.”

# Age



The screening process tracks patient's age, whether or not they're pregnant, and different drug allergies. This screen is the first of three questionnaires, and allows the screener to list whether each patient is pregnant, breast feeding, or under 8 years of age. In each case, the screener can list an answer as yes, no, or don't know. Each answer is taken into account by the device automatically to generate results. Answers to these questions are saved on the spot, so the screener can navigate anywhere during the process without losing progress.

# Doxycycline



This page queries whether the patients are allergic to doxycycline.

There are other drugs that fall within the confines of this allergy, so if the screener presses the “info” button, they are provided with the full list

# Ciprofloxacin



Finally, our last question asks whether the patients are allergic to ciprofloxacin.

After answering the question, selecting “Next” will bring the screener to the decision screen – where the application will make recommendations for patient care.

# Results



Now we have the results.


Robert should be given doxycycline, John should be referred to a physician, and Lisa should be given ciprofloxacin.

# Settings



By going to the menu and selecting “Settings,” you can specify a screener name, device id (shown with the record in the online database), “upload after” (once there are this many number of records created, the device will automatically ask to upload), upload url, and doxy and cipro lot numbers.

# Records

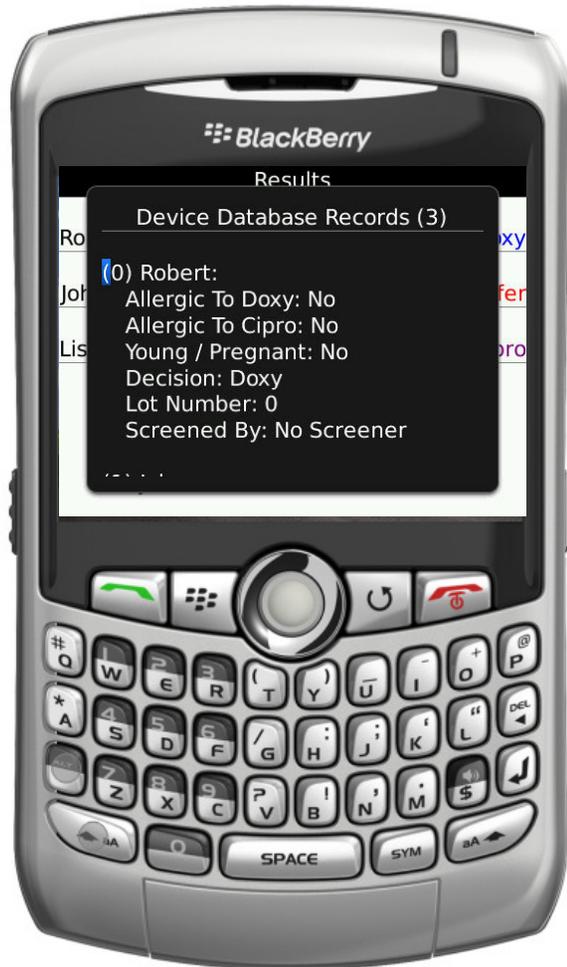

By going to the menu and selecting “View Records,” you will be able to view all the information about all of the records currently stored on the device (ones that were created since the last upload)

# Uploading



Finally, to upload records from the device to the online database that will track all of the system's records, go to menu and select "Upload." This will open a box that shows the progress of the upload, but this can be hidden and the screener can continue working while the upload takes place.